



ARIES®

ARIES® Systems | Welcome to the New Way to Work

ARIES® Exo+ Ready Mix




ARIES® Exo+ Ready Mix provides the key PCR ingredients needed to run user-defined protocols on any ARIES® System based on traditional hydrolysis probe technology. This mix comes in an individually sealed tube to which users can simply add target primers and probes, followed by direct attachment to an ARIES® Extraction Cassette. Features include:

- **Increased Lab Efficiency** – Lyophilized ready-to-use reagents eliminate the need to prepare and thaw
- **Broad Compatibility** – RNA and DNA targets can be detected using the same ready mix
- **Streamlined Workflow** – Reduction in hands-on time improves operational efficiency and offsets costs



Workflow

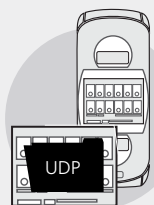


Step 1

Create the Assay Protocol File with the UDP Software and upload to the ARIES® System.




Step 2

Place the ARIES® Exo+ Ready Mix tube into the Sample Prep Tray. Add primers and probes as needed.

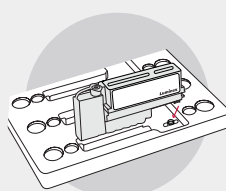

Step 3

Attach the ARIES® Extraction Cassette to the ARIES® Exo+ Ready Mix tube.

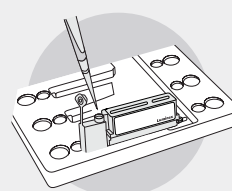

Step 4

Add the specimen to the cassette.



Step 5

Run the cassette on the ARIES® System using a user-defined Assay Protocol File.

Ordering Information

Product Name	Part Number
ARIES® Exo+ Ready Mix (Contains <i>Taq</i> polymerase)	3698 (24 tubes)
ARIES® Two Module System (IVD)	ARIES-M12V1-IVD
ARIES® M1 System (IVD)	ARIES-M6V1-IVD
SYNCT™ Software IVD	CN-SW47

Luminex®
complexity simplified.

support@luminexcorp.com or orders@luminexcorp.com

For Laboratory Use. Products are region specific and may not be approved in some countries/regions. Please contact Luminex at support@luminexcorp.com to obtain the appropriate product information for your country of residence.

©2018 Luminex Corporation. All rights reserved. Luminex, ARIES, and MultiCode are trademarks of Luminex Corporation, registered in the U.S. and other countries. SYNCT is a trademark of Luminex Corporation.

www.luminexcorp.com/exo+/

HEADQUARTERS

UNITED STATES

+1.512.219.8020

info@luminexcorp.com

EUROPE

+31.73.800.1900

europa@luminexcorp.com

CANADA

+1.416.593.4323

info@luminexcorp.com

CHINA

+86.21.8036.9888

info@luminexcorp.com

JAPAN

+81.3.5545.7440

infojp@luminexcorp.com

SS119653